

**DESCRIPTION:**

Treat the acne and soothe the skin at the same time. The Derasound Elite will help with the extraction process and helps to treat the inflammation that accompanies acne.

**CLEANSE**

1. Dispense 2 pumps Mega-Purifying Cleanser into dry hands and massage into dry skin for 2 minutes. Remove with tepid water.
2. Dispense a nickel size amount of Sal-X Purifying Cleanser into damp hands creating a lather. Massage into the skin for 2 minutes. Remove with tepid water. Spray Skin Recovery Mist onto the skin.

**TREAT**

3. Dispense 2 tsp Serious Action Masque and apply to the skin using a fan brush. Allow to remain on the skin for 5 minutes. Remove with warm towels.
4. Mist the skin with Skin Recovery Mist. Apply a light layer of Restorative Skin Clarifying Masque. Allow to remain on the skin for 3-5 minutes. Steam can be incorporated. Use warm towel to remove product.
5. Perform extractions with the Derasound on the Cavitation setting. Please refer to Derasound protocols for further instructions.
6. If additional extractions need to be performed, you may do this now. Finish off with a saturated cotton round with Skin Astringent No. 2 and keep skin clean.
7. Mix Sulphur Masque and Ultra Hydro Gel and apply with a fan brush. Allow to sit for 5 minutes. Remove with a cold towel and remove any residue with Skin Recovery Mist.
8. Using the Derasound on the Sonophoresis setting, apply all products to the skin without needing to absorb: Derma Pigment Skin Brightener, Skin Restoring Fulvic Elixir, Skin Medication No. 5, Retinol Restart Rejuvenation Serum and Ultra Hydro Gel.

**BALANCE**

9. Using the Derasound on the Microamp Setting perform your finishing touch. IF more slip is needed, apply 1 pump Ultra Hydro Gel.

**PROTECT**

10. Apply Photo-Age Environmental Protection Gel 15 and gently pat onto the face, neck and décolleté. Allow to absorb into the skin.

Recommended Treatment Time: 60 minutes